



TrioDocs

Version: 0.4.0

Date: May 25, 2025

Download the latest version on:
<https://triodocs.org>

Notifications

Notifications

Allows configuration of Trio notifications. Adjust based on your preferences.

Notifications can be enabled/disabled by toggling `Allow Notifications` on/off in `iPhone Settings > Notifications > Trio`.

Glucose

Show glucose on the app badge

This will add your current glucose on top of your Trio icon.

Always Notify Glucose

A notification will be triggered every time your glucose is updated in Trio.

Always play alert sound

This will cause a sound to be triggered by every Trio notification.

Also add source info

The source of the glucose reading will be added to the notification.

Low

Set this to the highest value you want a low glucose notification to be triggered at.

High

Set this to the lowest value you want a high glucose notification to be triggered at.

Carbs Required Threshold

Trio may predict a low blood sugar event and recommend consuming a suggested amount of carbs. `Carbs Required Threshold` determines the minimum amount of carbs required before Trio sends a notification.