



TrioDocs

Version: 0.4.0

Date: May 25, 2025

Download the latest version on:
<https://triodocs.org>

iPhone

Compatible Phones

Trio requires a relatively recent iPhone. The faster the phone, the better for the algorithm and battery life. You need a minimum version of the mobile operating software, called the phone's "iOS", to be installed on your iPhone. Trio is compatible with iPhones running on iOS 17 or newer.

iPhone

Trio is compatible with phones running on iOS 17 or newer.

These iPhones are compatible with iOS version 17 and newer:

- iPhone 16, all variants
- iPhone 15, all variants
- iPhone 14, all variants
- iPhone 13, all variants
- iPhone 12, all variants
- iPhone 11, all variants
- iPhone XS Max
- iPhone XS
- iPhone XR
- iPhone SE (3rd generation; 2022 first release)
- iPhone SE (2nd generation; 2020 first release)

iOS Version

If your iOS version is not listed, e.g., 17.6.1, choose the first row that is less than your iOS version.

<u>iOS</u> Version	Minimum <u>Xcode</u> Version	Minimum <u>macOS</u> Version
18.1	16.1	14.5
18.0	15.4	14.5
17.5	15.4	14.0
17.4	15.3	14.0
17.0	15.0	13.5

Warning

Your phone's iOS dictates your macOS requirements

The more up-to-date you keep your phone's iOS, the more up-to-date your computer and macOS must be to build Trio with Xcode.

iOS Version and Developer Mode

Find Your iOS Version

Your phone's iOS version can be found under the phone Settings -> General -> About display. The iOS number is found on the Software Version line. When you start looping, you should delay iOS updates until others have confirmed the software functions as normal on the newest update.

Warning

Using iOS Beta versions on the iPhone used for insulin delivery is **never** recommended.

Developer Mode - Mac Build

With iOS 16 and watchOS 9, Apple added a feature called "Developer Mode." This feature must be enabled on your iPhone and Apple Watch to allow the installation of Trio. To learn more, click on this link: [Apple Link about Developer Mode](#).

When you build Trio on your phone from Xcode directly, you must enable Developer Mode. This is also required in order to use the Trio app on a watch paired to your phone.

Developer Mode with iOS 16, watchOS 9

If you already have Trio, built with Xcode on a Mac, on your phone/watch, when you update to iOS 16/watchOS 9, you will be told that Trio requires Developer Mode to run.

You cannot run Trio on your phone (or watch) until you have enabled Developer Mode on the device(s).

Automatic Updates

Apple provides frequent updates to the iOS. These updates often include critical security patches and improved new features.

Updates to iOS could make Trio stop working.

- **Minor** updates (like 16.5 -> 16.5.1) are usually not an issue.

- **Major** updates (like 17.x -> 18.x) often contain changes that Trio developers need to include in the app.

Turn Off Automatic Update Installs!

- Once you accept an iOS update, you cannot go backward
- Some iOS updates require updates to Xcode and macOS before people can build Trio on that device again
- It is rare, but iOS updates have caused Trio to stop working until other updates were made and Trio was rebuilt on that phone
- Turn off automatic updates so you can choose when to update your phone and avoid being caught without a working Trio app
- Google the instructions for your device:
 - 1) Configure your phone to *download* the updates automatically
 - 2) Choose to perform the *installation* of the updates manually

Battery Health

Make sure your phone's battery is functioning optimally. Your phone will become a critical health device, and you want it to keep working.

- Make sure a charger and cord are in your diabetes supplies
- Consider buying a battery pack, keep it charged, and add it to your travel bag

Low Power Mode

Some people have reported that Trio continues working in the background (phone locked) even in **Low Power Mode**. Others have reported that they still get red loops. You can experiment to determine if your phone/iOS/app is able to maintain green loops in low-power mode. Otherwise, the best practice is to avoid Low-Power Mode.

Apple Watch

Trio has a companion app for the Apple Watch. The companion app shows the loop status and current insulin and carbs onboard. It also lets you add carbs, do a manual bolus, and set a temporary target. However, the companion app only serves as a remote control for the iPhone app; connection to the CGM and pump is still done through your iPhone, and you will have to bring it with you.

Apple Watch Series 3 and newer are supported. Newer versions of the Apple Watch have better battery life and support always-on display.